

Validity 3 years

Packing OPP bag

Storage conditions Well ventilated, dark and dry environment



Date of preparation 16th March, 2020

Date code See the packing

Manufacturer

Medical Device Branch of Zhangzhou Easepal Industrial Co., Ltd

Address


4th Floor of Building #7, No.228, Jiaosong Road, Taiwanese Investment Zone, Zhangzhou, Fujian, China

Postcode 363107

EASEWELL™

Personal protective mask

INSTRUCTION FOR USE

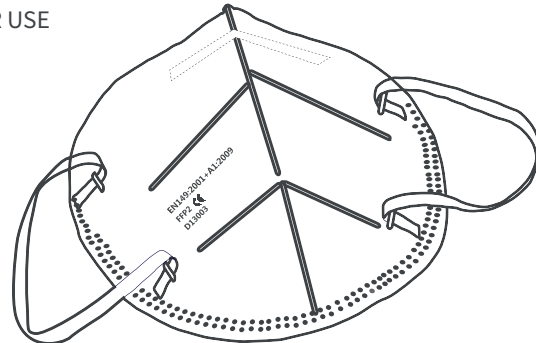

Product name Personal protective mask(non-sterile conditions)

Product Specification folded size 106.7mm×155mm

Structure composition total 3 layers: 2 layers non-woven fabric and melt-blown fabric

Scope of use Mainly for medical and health units and family health care.

Instruction For Use



1. Hold respirator with nose piece on top and pull open (as shown).



2. Place chin and face in mask with nose piece on the bridge of your nose.



3. Stretch and position the band on your ears.



4. Using the band, mold metal nose piece to shape of nose for tight fit.



5. To test fit, cup both hands over the respirator and exhale vigorously. If air flows around your nose. Tighten the nose piece. If air leaks around the edges, re-position the straps for better fit.

Warning

1. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
2. A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use.
3. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%.
4. Do not use when concentrations of contaminants are immediately dangerous to health or life.
5. Do not use in explosive atmospheres.
6. Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs.
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of the respirator.
8. Never alter or modify this respirator in any way.
9. Only for single use. No maintenance necessary. Discard the used respirator after single use.
10. Keep respirators in the display box away from direct sunlight until use.